Erkodur-0M1

CE 93/42/EWG

DIN EN ISO
13485 : 2012
9001 : 2008



Erkodur-0M1

Erkoform-3/3d(+)/3d motion
Erkopress 300 Tp/-ci

Erkoform-RVE
Erkopress ES 200 E

			
0,6 mm, 170 °C	0:30 min	0,6 mm, 0:40 min	
1,0 mm, 165 °C	0:45 min	1,0 mm, 0:50 min	
2,0 mm, 155 °C	1:30 min	2,0 mm, 1:25 min	

Lagerung • storage • stockage

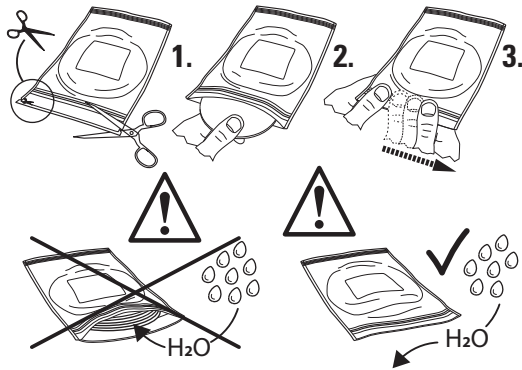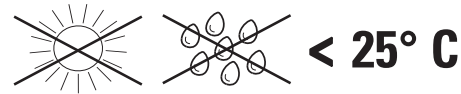ERKODENT Erich Kopp GmbH
Siemensstraße 3
72285 Pfalzgrafenweiler, Germany
Tel. + 49 (0) 74 45/85 01-0
Fax + 49 (0) 74 45/20 92
info@erkodent.com • www.erkodent.com

BA 45-0M1-02-16

DE Erkodur-0M1

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EN Erkodur-0M1

Chemical characteristics:

PETG, copolyester • Insulating foil: PE, polyethylen.
Harmless to health, BgVV (Federal Health Office) listed
thermoplast with tested biocompatibility. Allergic reactions
are improbable but cannot be excluded. Insoluble in water,
inactive, harmless to ground water.

Disposal/Recycling: General recycling for
plastic/household waste.

Field of use:

(see brochure thermoforming technique and material card)
Occlusal splints, bruxism splints,
stabilization splints, dressing plates, correction splints
(aligners), cosmetic splints. Adjustment is possible with
the Occluform (-3), by grinding or by addition.

Material properties:

Viscous hard foil. Extremely impact-resistant.
Bonds to acrylate (Resilit-S, 817 501 + 817 503).

Attention, sensitive to humidity: Immediately after
having taken a plate reclose the aluminium bag and keep
it closed.

Finishing: (see brochure thermoforming technique)

Finishing set Quick 2 (110 877), high polish will be obtained
with polishes for plastic. The spacer foil will be taken off
only after thermoforming and finishing. Moisten the area
that has to be adjusted with acrylate at first with little
monomer.

Plastification: Place the foil in the devices in such a way
that the spacer/insulating foil is pointing towards the
model. For machines without control: Test softness of foil
with an instrument. If permanent impressions result,
thermoform.

Availability, Erkodur-0M1, tooth-colored
Ø 120 mm

thickness 0.60 mm, 20 pcs., **52 61 06**
thickness 1.00 mm, 20 pcs., **52 61 10**
thickness 2.00 mm, 10 pcs., **52 61 20**

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