
# UZF-Plus

DIN EN ISO  
13485: 2003

Lagerung • storage • stockage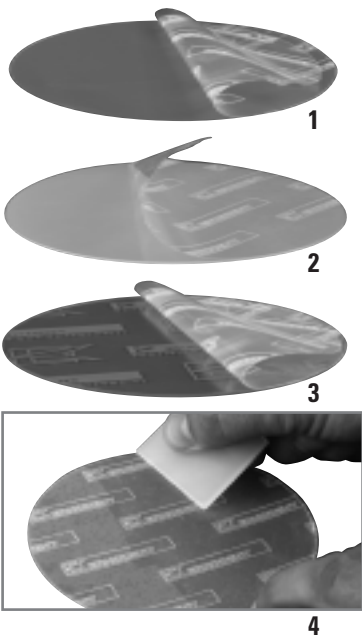

**ERKODENT Erich Kopp GmbH**

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72285 Pfalzgrafenweiler, Germany

Tel. + 49 (0) 74 45/85 01-0

Fax + 49 (0) 74 45/20 92

info@erkodent.com • www.erkodent.com

# DE UZF-Plus

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PE, Polyethylen: UZF-Plus Platzhalter-/Isolierfolie

PS, Polystyrol: Trägerfolie

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PE, polyethylene: UZF-Plus spacer/insulating foil

PS, polystyrene: carrier foil

Harmless to health, BgVV (Federal Health Office) listed thermoplasts with tested biocompatibility. Insoluble in water, inactive, harmless to ground water.

## Disposal/Recycling:

UZF-Plus: general recycling for plastic/household waste (combustible, low in pollutants and high in energy).

Carrier foil: If possible recycling with polystyrene (PS) of same nature.

## Field of use:

(see brochure thermoforming technique and material card)

UZF-Plus can be applied by one or two layers as spacer/insulating foil on each Erkodent foil. Also additionally (then only one layer) on already ex works applied spacer/insulating foils.

As spacer: The shrinkage of the thermoforming material is compensated. Splints fabricated in this way fit better and sit on the teeth row with less tension.

As insulating foil: Rests of silicones, wax and other rests on the plaster model remain reliably separated from the inner side of the foil. This saves tedious cleaning. The inside of the foil will be hygienically smooth.

## Material properties:

Transparent, 0,1 mm thick foil with white logo. Will always be applied towards the model and is heated with the basis foil.

The heating times and rated temperatures of the basic foils do not change, **aside from Erkoflex.**

For Erkoflex please see the indications of: Erkoflex + Universal-UZF or Erkoflex + UZF-Plus.

## Working hints:

Remove UZF-Plus of the carrier foil, picture 1.

Rub on UZF-Plus on the basis foil, picture 2.

Rub on UZF-Plus additionally on existing spacer/insulating foil, picture 3.

Remove air incorporations with enclosed plastic rectangle (PE) as good as possible, picture 4.

## Plastification: (for machines without heating time control)

Soft foils: Foil slightly sags and tenses again, then thermoform.

Hard foils: Check softness with an instrument. If permanent impressions result, thermoform.

## Availability:

UZF-Plus on carrier foil, clear with logo, Ø 120 mm

thickness	contents	order number
0.1 mm	50	53 12 01

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