## Wasserbad

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## Waterbath

**Voltage and type** see identification plate on the appliance. The appliance complies with the Guidelines no. 89/336/EEC for electromagnetic compatibility and no. 73/23/EEC for safety.

### Instructions for use:

#### Important safety hints

Connect only 230 V ac (safety socket). Completely unwind the supply. **Caution:** The outside of the housing gets hot. Danger of scalding through escaping steam. Do not operate the appliance without any liquid, boiling dry should be avoided. Never immerse the appliance in water. Always put the appliance on a heat-resistant and even surface. Only operate the appliance under supervision. If it is not used, separate the appliance from the mains.

#### Operation

Take off the lid and fill in the desired quantity of water (max. 1.5 l). **While heating put the immersion cradle in that way into the container that the bow looks out of the pouring-out-opening and close the lid.**

Provide power supply, the appliance is ready for operation, the upper indicator lamp shines (position „min“ of the rotary type regulator). Adjust temperature at the rotary type regulator. The heating is automatically connected (the lower indicator lamp shines) until the chosen temperature is reached or will be maintained.

Always plastify materials that have to be heated up in the immersion cradle.

#### Run-dry protection/overheat protection

If the appliance is used inappropriately or is strongly calcified, it is automatically de-activated by the run-dry protection. If the run-dry protection is actuated pull the mains plug and allow the appliance to cool down for a few minutes. Only when the appliance has cooled down refill with water, otherwise injuries through scalding are possible.

#### Cleaning and care

Pull the plug before any cleaning. Never immerse the appliance in water. Never use strong chemicals or abrasives. To remove slight occurrences of calcification fill the device with 0.5 ltrs of water, add 5 to 6 tablespoonsful of vinegar and bring to the boil. Decalcification of the appliance from time to time helps to keep power consumption low to prolong its service life.

#### Technical specifications: 1800 W, 230 V~

Interior diameter of the pot: 140 mm

Usable height without/with immersion cradle: 130 mm/110 mm

The adjusted temperature will be maintained with +/- 4 °C.

Ord. no.: **120 300** Waterbath with immersion cradle

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## 2 years warranty

For this product we provide warranty for 2 years from date of purchase against fabrication or material defects. The warranty begins with the day when the appliance has been handed over and is only valid **together with the invoice or delivery note**. This warranty does not apply to damage arising through normal wear and tear, improper treatment and failure to observe the operating instruction. Any complaints exceeding the assured warranty will not be recognised by ERKODENT. Any complaints may not be remedied by persons not expressly authorised so to do. All claims will become extinct should complaints be remedied in the above manner.

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