# Erkodur-0M1

CE 93/42/EWG

DIN EN ISO  
13485 : 2012  
9001 : 2008


## Erkodur-0M1

Erkoform-3/3d(+)/3d motion  
Erkopress 300 Tp/-ci

Erkoform-RVE  
Erkopress ES 200 E

°C	☀️🕒	☀️🕒
0,6 mm, 170 °C	0:30 min	0,6 mm, 0:40 min
1,0 mm, 165 °C	0:45 min	1,0 mm, 0:50 min
2,0 mm, 155 °C	1:30 min	2,0 mm, 1:25 min

## Lagerung • storage • stockage

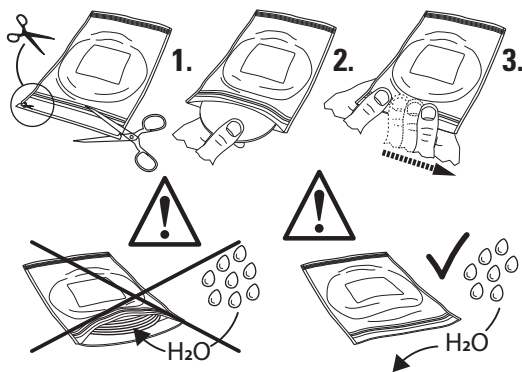## ERKODENT Erich Kopp GmbH

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72285 Pfalzgrafenweiler, Germany

Tel. + 49 (0) 74 45/85 01-0

Fax + 49 (0) 74 45/20 92

info@erkodent.com • www.erkodent.com

BA 45-0M1-02-16

# DE Erkodur-0M1

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# EN Erkodur-0M1

## Chemical characteristics:

PETG, copolyester • Insulating foil: PE, polyethylen. Harmless to health, BgVV (Federal Health Office) listed thermoplast with tested biocompatibility. Allergic reactions are improbable but cannot be excluded. Insoluble in water, inactive, harmless to ground water.

**Disposal/Recycling:** General recycling for plastic/household waste.

## Field of use:

(see brochure thermoforming technique and material card) Occlusal splints, bruxism splints, stabilization splints, dressing plates, correction splints (aligners), cosmetic splints. Adjustment is possible with the Occluform (-3), by grinding or by addition.

## Material properties:

Viscous hard foil. Extremely impact-resistant. Bonds to acrylate (Resilit-S, 817 501 + 817 503).

**Attention, sensitive to humidity:** Immediately after having taken a plate reclose the aluminium bag and keep it closed.

**Finishing:** (see brochure thermoforming technique) Finishing set Quick 2 (110 877), high polish will be obtained with polishes for plastic. The spacer foil will be taken off only after thermoforming and finishing. Moisten the area that has to be adjusted with acrylate at first with little monomer.

**Plastification:** Place the foil in the devices in such a way that the spacer/insulating foil is pointing towards the model. For machines without control: Test softness of foil with an instrument. If permanent impressions result, thermoform.

**Availability, Erkodur-0M1, tooth-colored**  
Ø 120 mm

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