Erkodur

DIN EN ISO
13485 : 2012
9001 : 2008

Erkodur, klar, clear, clair • Erkodur, freeze

Erkoform-3/3d/3dmotion/+
Erkopress 300 Tp/-ci

Erkoform-RVE
Erkopress ES 200 E

Stärke/thickn. épaisseur	Temperatur	Dauer
0,5 mm	160 °C	0:30 min
0,6 mm	160 °C	0:30 min
0,8 mm	160 °C	0:45 min
1,0 mm	160 °C	0:45 min
1,5 mm	160 °C	1:00 min
2,0 mm	155 °C	1:30 min
3,0 mm	155 °C	2:00 min
4,0 mm	155 °C	3:00 min
5,0 mm	155 °C	4:00 min

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EN Erkodur, clear • freeze • -A1 • -A2 • -A3

Chemical characteristics: PETG, copolyester • Insulating foil: PE, polyethylen. Thermoplast harmless to health, with tested biocompatibility. Allergic reactions are improbable but cannot be excluded. Insoluble in water, inactive, harmless to ground water.

Disposal/Recycling: General recycling for plastic/household waste.

Field of use: (see brochure thermoforming technique and material card) Occlusal splints, bruxism splints, stabilization splints, dressing plates, correction splints (aligners), moulded appliances. Silensor-sl. Adjustment is possible with the Occluform (-3), by grinding or by addition.

Material properties: Viscous hard foil. Extremely impact-resistant. Bonds to acrylate (Resilit-S, 817 501 + 817 503).

Attention, sensitive to humidity: Immediately after having taken a plate reclose the aluminium bag and keep it closed. **Finishing:** (see brochure thermoforming technique) Finishing set Quick 2 (110 877), high polish will be obtained with polishes for plastic. The spacer foil will be taken off only after thermoforming and finishing. Moisture the area that has to be adjusted with acrylate at first with little monomer.

Plastification: Place the foil in the devices in such a way that the spacer/insulating foil is pointing towards the model. For machines without control: Test softness of foil with an instrument. If permanent impressions result, thermoform.

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