ERKOLOC

CE 93/42/EWG

DIN EN ISO
13485: 2003

Erkoform-3/3d
Erkopress 300 Tp/-ci

1,8 mm, 135 °C
3,0 mm, 120 °C

1,8 mm, 1:15 min
3,0 mm, 1:45 min

Erkoform-RVE
Erkopress 200 E

1,8 mm, 1:20 min
3,0 mm, 2:05 min

Lagerung • storage • stockage

ERKODENT Erich Kopp GmbH

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72285 Pfalzgrafenweiler, Germany

Tel. + 49 (0) 74 45/85 01-0

Fax + 49 (0) 74 45/20 92

info@erkodent.com • www.erkodent.com

BA 40-0104-2

DE ERKOLOC

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EN ERKOLOC

Chemical characteristics:

SBS, hard side: styrol-butadien-styrol
EVA, soft side: ethyl-vinyl-acetate
PE, insulating foil: polyethylene
Harmless to health, BgVV (Federal Health Office)
listed thermoplasts with tested biocompatibility.
Insoluble in water, inactive, harmless to ground
water.

Disposal/Recycling:

Household waste (combustible, low in pollutants and
high in energy)/general recycling for plastic.

Field of use: (see brochure Thermoforming tech-
nique and material card) Comfortable stabilization
and occlusal splints.

Material properties:

Double layer plate, hard/soft. Hard layer adjustable
by addition using acrylics (Resilit-S, 817 501 +
817 503). Do not move splint for app. 2 hours after
building-up with acrylics to ensure a stable bond.
Limited durability, not suitable for permanent splints
(recommendation for permanent splints:
ERKOLOC-pro).

Finishing:

(see brochure Thermoforming technique) Finishing
set Quick 2 (110 877). The spacer foil will be taken
off only after thermoforming and finishing.

Plastification:

(for machines without heating time control) The hard
side of the ERKOLOC plate should be heated. Place
the foil in the devices in such a way that the spacer/
insulating foil is pointing towards the model. Test
softness of foil with an instrument. If permanent
impressions result, thermoform.

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